# Mikromotor 31S

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# Micromotor 31S

OPERATING INSTRUCTIONS

No. 18400 V36/15

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## Safety measures GB

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Nouvag AG can not be held liable for any malfunction of this device performance failure, and/or its designed utility, nor can Nouvag AG be held liable for any injuries, in any case when the device is missused or not operated, applied or maintained in strict accordance with the user/owner instructions set out in the Operating Manual!

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## Cleaning, Disinfection and Sterilization GB

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- Do not bend the motor cable!
- Clean, disinfect and sterilize the Micromotor after each use!
- Do not clean Micromotor 31 with compressed air!
- Autoclaving the electronic motor without packing it into a transparent bag may seriously damage it!
- Do not fill the sterilization bag more than 80 %!

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- 1) The micromotor, cable and plug should be cleaned after each use to prevent the build up of deposits and debris that can destroy it, if not removed. Wipe it with a clean cloth dampened with disinfectant solution.
- 2) Unscrew the handpiece carrier.
- 3) Clean handpiece carrier as well as handpiece carrier inner part.
- 4) Screw handpiece carrier, again.
- 5) Pack the motor into a sterilization bag (bags should be conform to DIN 58953).
- 6) After packing motor and cable, autoclave it at maximum 135°C.
- 7) After autoclaving, allow motor to dry for one hour at room temperature.

Store motor bagged and tagged with sterilization date!

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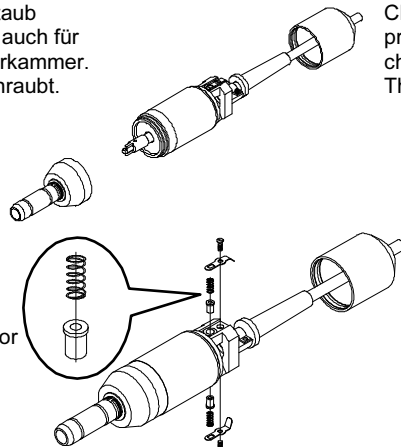
### Cleaning the cavities:

Remove weekly carbon and grinding dust with the brush!

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Clean with the enclosed brush the motor chamber. The same procedure applies for the inner part of the handpiece carrier chamber.

The chamber lids can be respectively screwed or unscrewed.

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### Changing the carbon brushes:

- 1) Unscrew motor cover.
- 2) Remove the screws on the side with the appropriate screwdriver.
- 3) Remove worn out carbon brushes and springs.
- 4) Insert new carbon brushes and springs  
→ curved carbon brushes end must rest on the collector.
- 5) Tighten screws on the side, again.
- 6) Screw motor cover, again.

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Problem:	Cause:	Solution:
Motor does not run	Plug is not inserted properly	Insert plug and check fitting
Motor stops when cable moves	Defective cable	Replace cable
Motor stops occasionally	Carbon brushes worn out	Replace brushes
Motor runs, but tool does not turn	Handpiece is not connected properly to the motor	Press handpiece firm on motor until handpiece snaps in

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Weight: ..... 199 g  
 Max. output: ..... 80 W  
 Max current: ..... 4 A  
 Rated voltage: ..... 30 V  
 Rated speed: ..... 35,000 rpm  
 Micromotor coupling: ..... according to EN23964  
 Cable length: ..... 2m / 4m

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Art.-Nr.	Description	Units
1115	Carbons brushes with spring	two pieces each
1198	Brush	1 piece

**Servicestellen****Service Centers****Switzerland:**

Hersteller / Manufacturer:  
 NOUVAG AG  
 St. Gallerstrasse 23-25 • CH-9403 Goldach • Switzerland  
 Phone: ++41 (0)71 846 66 00 • Fax: ++41 (0)71 845 35 36  
 E-mail: info@nouvag.com • www.nouvag.com

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**All NOUVAG-Service Centers worldwide pls. see at  
[www.nouvag.com](http://www.nouvag.com)**

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**Disposal:**

Disposal of device, components and accessories must strictly conform to local laws and regulations as set out by the relevant authorities.

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NOUVAG GmbH
NOUVAG USA Inc.