# Motorhandstück 38

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**NOUVAG**® SWISS  
Dental and Medical Equipment QUALITY

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- Nouvag AG can not be held liable for any malfunction of the device performance failure, its designed utility and/ or when the device is miss-used or not operated in accordance with the user instructions set out in Operating Manual.
- When storing the instrument without a clamped tool for long periods of time, the chuck must be unclamped!
- Run hand piece only with clamped tool!

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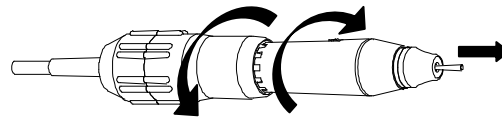
Do not remove the chuck while the Motor is running!

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The chuck 2.35 mm is usually included with the motor handpiece. The chuck can be opened and closed by twisting the hanpiece front part and back part:

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

- ...clockwise ..... to open chuck  
→ grinder can be changed
- ...counterclockwise ..... to close chuck  
→ grinder fixed

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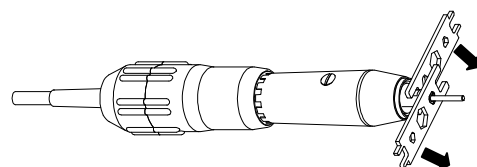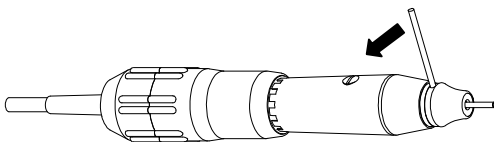- 1) Closed chuck with inserted dowel pin  $\varnothing$  2.35mm (included)
- 2) Insert the end of the pin  $\varnothing$  2.5mm (included) into the hole and loosen tip by pressing counterclockwise.

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**Without clamped dowel pin, the chuck can get damage and become deformed due to pressure!**

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- 3) Loosen chuck with help of the 2 keys (included) → hold shaft firmly with one key and unscrew chuck with the other one.
- 4) Insert and tighten new chuck

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Assembling after the change is similar, but in reverse order and counterclockwise

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- Motorhandpiece 38 is not sterilizable!
- Do not clean Motorhandpiece 38 with compressed air!
- Do not bend the motor cable!

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The Motorhandpiece, cable and plug should be cleaned after each use to prevent the internal build-up of deposits and debris that can destroy it, if not removed. Use the brush and wipe it with a clean, damp cloth.

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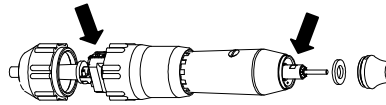
**Remove weekly carbon and grinding dust with the brush!**

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
Clean with the enclosed brush the soiled motor chamber. The same procedure applies for the chuck chamber. **The dusty felt packing in the chuck chamber must be cleaned and replaced when necessary.** The tip and motor cap can be screwed or unscrewed respectively

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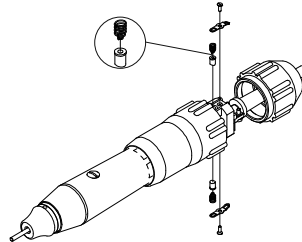**Changing the carbon brushes:**

- 1) Unscrew motor cover
- 2) Remove the screws on the side with the appropriate screwdriver
- 3) Remove wore out carbon brushes and springs
- 4) Insert new carbon brushes and springs  
→ **curved carbon brushes end must rest on the collector**
- 5) Tighten screws on the side, again
- 6) Screw motor cover, again

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Problem:	Cause:	Solution:
<b>Motor does not run</b>	Plug is not inserted properly	Insert plug and check fitting
<b>Motor stops when cable moves</b>	Defective cable	Replace cable
<b>Sporadic motor disfunction</b>	Worn out brushes	Replace brushes

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Weight:..... 375 g  
 Max. output: ..... 80 W  
 Max current: ..... 4 A  
 Rated voltage: .....30 V  
 Rated speed:..... 35,000 rpm  
 Cable length.....2 m  
 Chuck diameter standard:.....2,35 mm  
 Chuck diameter optional..... 1,6 / 3 / 3,175 mm

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Art.-Nr	Description	Units
1115.....	Carbons brushes with spring .....	two pieces each
1198.....	Brush.....	1 piece
20593.....	Key for chuck.....	2 pieces
20572.....	Felt packing.....	1 piece
20165.....	Dowel pin Ø 2.5mm.....	1 piece

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**www.nouvag.com**

**All NOUVAG-Service Centers worldwide pls. see at**

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**Disposal:**

Disposal of device, components and accessories must strictly conform to local laws and regulations as set out by the relevant authorities.

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NOUVAG AG
NOUVAG USA Inc
NOUVAG GmbH