DE Gültig ab Mai 2013

Revision: 2.0, Deutsch

WICHTIGE PRODUKT-INFORMATION

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Revision: 2.0/English

IMPORTANT INFORMATION

PLEASE READ BEFORE USE. FAILURE TO READ AND FOLLOW THESE INSTRUCTIONS MAY RESULT IN SERIOUS PATIENT INJURY OR DEATH:

INDICATIONS

The Nouvag disposable Perforator „NouvaCut Single Use“ is for use in perforating the cranium. When properly used, it is designed to automatically disengage once perforation is accomplished and when pressure is removed from the drill point.

DISRIPTION

The Nouvag disposable Perforator „NouvaCut Single Use“ is a single use device. It is a disposable perforator with a Hudson end and is available in four color-coded sizes:
Item number: 1977; 1976, 11 mm, (purple)
Item number: 1978; 1920, 9 mm, (gray)
Item number: 1979; 1921, 13 mm, (blue)
Item number: 1980; 1922, 14 mm, (orange)

All Nouvag disposbale Perforators with the item number 1978, 1977, 1979, 1980 are designed for usage in areas with thin bone.

All Nouvag disposable Perforators with the item number 1920, 1976, 1921, 1922 are designed for usage in areas with thick bone. Caution should be taken to avoid use in children and/or infants.

RECOMMENDATION

A second sterile Perforator or alternative method must always be available as backup in the event sterile technique is compromised. Any resterilization of this device can cause malfunction resulting in serious patient injury or death.

WARNINGS ⚠

It is extremely important that this section be read carefully before attempting to use the perforator.

- The surgeon must always be aware of the possibilty that the perforator may not disengage.
- Exercise extreme caution when the perforator nears the point of complete perforation. The underlying bone may not be sound and can fracture if excessive pressure is used, causing the perforator to penetrate the brain.
- The perforator can nick or tear dura or brain at the time of disengagement if conditions such as adherent dura, high intracranial pressure, or other underlying abnormalities exist in the area of perforation.
- The perforator can nick or tear dura or brain if it is not held perpendicular to the skull at the point of perforation.
- Excess deviation from the perpendicular can cause malfunction that could result in patient injury or death.
- The Nouvag disposable Perforator „NouvaCut Single Use“ is designed for perforating normal, healthy cranial bone. Exercise extreme caution when performing skull perforators on infants, children, the elderly, and diseased bone because of the varying consistencies and thicknesses of bone.
- Nouvag AG does not recommend use of the disposable perforator in a previously drilled burr hole or in surrounding areas. The composition of the restoration differs from healthy bone, thereby increasing the risk of penetration into the brain.
- Exercise extreme caution when making perforations in thin bone (< 1.75 mm) such as the temporal bone.
- If the perforator is stopped or prematurely disengages in the middle of a burr hole, there is the possibility that it may not reengage when attempting to drill again in the same burr hole.
- The Nouvag disposable Perforator „NouvaCut Single Use“ (fig. ❶) is to be used for one procedure only. Resterilization by any method could cause a malfunction that could result in patient injury or death. Do not use if perforator has been reesterilized by any means.
- Do not use the perforator if the label is damaged.
- The NouvaCut perforators may be operated up to a speed of 1200 revolutions per minute.
- Power unit must be used only in the clockwise direction.

PRECAUTIONS

Carefully inspect the sterile package.

Do not use the product if:

- the package or seal appears damaged,
- contents appear damaged, or
- the expiry date has passed.

STERILITY

This product is for SINGLE USE ONLY; DO NOT RESTERILIZE. Use aseptic technique in all phases of handling. Do not reuse. Discard after one procedure. Nouvag AG will not be responsible for any product that is reesterilized, nor accept for credit or exchange any product that has been opened but not used. As long as the individual package is not opened or damaged, the product is sterile.

TESTING PROCEDURE

PERFORM THE FOLLOWING TEST PROCEDURE BEFORE EACH PERFORATION. FAILURE TO CONDUCT THIS PROCEDURE COULD RESULT IN SERIOUS PATIENT INJURY OR DEATH.

CAUTION

Place gauze between the perforator point and thumb when performing the test to eliminate the possibility of puncturing or tearing the glove.

- Visually examine the perforator and its packaging for any obvious defects, damage, or distortion. Read and inspect the labeling on the perforator.
- With adequate pressure applied on the perforator point, engagement should occur as the Hudson End is rotated. (fig. ❷). If engagement does not occur, do not use perforator.
- When engagement occurs, thumb pressure on the perforator point should produce a smooth, positive spring action. (fig. ❸). If smooth, positive spring action does not occur, do not use perforator.
- The Hudson End should rotate smoothly within the perforator body when the unit is in the disengaged position. (fig. ❹). If the Hudson End does not rotate smoothly, do not use the perforator.

GENERAL OPERATING PROCEDURE

- Insert perforator securely into driver.
- Press the perforator on the skull.
- Manually rotate the Hudson Snap-Lock on the driver until the perforator is in the engaged position.
- With the driver and attached perforator positioned perpendicular to the skull, apply power and maintain constant but controlled pressure until disengagement occurs.
- Repeat steps 1, 2 and 3 of the testing procedure and steps 2, 3 and 4 of the general operating procedure for each perforation.

LIMITED WARRANTY

Nouvag AG warrants this product to be free from defects in material and workmanship. If this product fails to meet this warranty, Nouvag AG will, at its discretion, replace it or refund the purchase price. This warranty will be void where the product has not been used in the manner directed or has been subject to accident or abuse. Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer’s instructions for use. There are no warranties that extend beyond the description on the face hereof.

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